



# MEDISEPT

Perfect disinfection



## MEDISEPT Wipes for hand and surfaces disinfection

### High-quality wipes for disinfecting hands and all small alcohol-resistant surfaces

Efficient wipes for disinfecting hands and surfaces, including surfaces of medical devices. It has bactericidal, yeasticidal, mycobactericidal, tuberculocidal, and limited virucidal effect (e.g. Herpes Simplex, HIV, Hepatitis B and C, Rota virus).

#### Product features:

- Tested in accordance with new standards for disinfectants.
- Hygienic disinfection in 30 sec. (soaking liquid tested in acc. with EN 1500).
- High quality, non-woven fabric: 50 g/m<sup>2</sup>.
- The capacity of one wipe is 0.5 m with a double rubbing.\*.
- Wipe dimensions: 19 x 15 cm.

\* The test was carried out based on the methodology of the EN 16615 standard for a four-field test, on a porous surface, at 24 ° C

#### User benefits:

- Recommended for hands and small areas in the medical area and in public places.
- Guaranteed effectiveness and safety of use.
- Delicate, fresh fragrance.
- Contain a moisturizer - glycerin.
- Closing made of hard plastic guarantees a constant moisture after opening.



Active against  
Tbc, Rota  
in 30 sec.



50 pcs packaging



Disinfects  
and protects  
hands



Disinfects  
surfaces

# MEDISEPT Wipes for hand and surfaces disinfection

## Application:

MEDISEPT Wipes are recommended for surfaces such as tabletops, knobs, handles, seating surfaces, medical equipment, and accessories made of plastic, stainless steel, porcelain, and ABS plastic. Due to its intended use on the skin, the product contains care-substance additives which moisturize the hands.

Effectiveness	Norm according to EN14885	Time
Bacteria (including MRSA)	EN 13727	30 sec.
Bacteria (including MRSA)	EN 16615	30 sec.
Fungi (C. albicans)	EN 13624	30 sec.
Fungi (C. albicans)	EN 16615	30 sec.
M. Tuberculosis (M. terrae)	EN 14348	30 sec.
Enveloped viruses (HIV, HBV, HCV, Vaccina, Influenza, Vaccinia, BVDV, Herpes simplex, Ebola, SARS-Cov-2)	EN 14476	30 sec.
Rota Virus	EN 14476	45 sec.
Hygienic disinfection*	EN 1500	30 sec.

\* test performed on saturated formulation,  
1 tissue contains approx. 3.56 ml of disinfecting liquid

## Directions for use:

Take a wipe out of the package and wipe hands or surface. Close the package tightly. After opening, use the wipes within 21 days. Do not use if the wipe is dry. Do not use on areas sensitive to alcohol.

## Ingredients: 50 pieces of tissues contain:

178 g of saturated formulation.

## 100 g of formulation contains:

60 g - Propan-2-ol,

0.5 g - d-gluconic acid, compound with N,N''-bis(4-chlorophenyl)-3,12-diimino-2,4,11,13- tetraazatetradecanediamidine (2:1).

## Packaging unit:



50 wipes in a flow pack with self-adhesive closing


## Permission number for biocidal products in Poland: 6678/16

Medical device class II within the meaning of the Medical Devices Act.

Intended for disinfecting non-invasive medical devices and for hygienic hand disinfection.

The safety data sheet and the component data sheet are available at [www.medisept.pl](http://www.medisept.pl)

CE 2274

 Producer  
**MEDISEPT Sp. z o.o.**  
Konopnica 159 c  
21-030 Motycz

phone: + 48 81 535 22 22  
[export@medisept.pl](mailto:export@medisept.pl)  
[www.medisept.pl](http://www.medisept.pl)

Revision: 04.2020