



MEDISEPT

Perfect disinfection



Velodes® Silk

Liquid for hygienic and surgical hand disinfection
and for surface disinfection

Product features:

- Active against influenza and coronavirus.
- Hygienic hand disinfection: 2 x 3 ml in 30 sec.
- Surgical hand disinfection: 2 x 3 ml in 3 min.
- Surface disinfection from 15 sec.
- Gentle pH.

User benefits

- Guaranteed effectiveness.
- Moisturizes and cares for the hands.
- Convenient for use.
- Rapid effect.
- Prevents hand stickiness.



Liquid



Active against
B, F, V, Tbc,
Rota, Noro



Hygienic hand
disinfection
in 30 sec.



Surgical hand
disinfection
in 3 min.

Application:

Ready to use, alcohol - based liquid for hygienic and surgical hand disinfection in the medical field, the food sector, industrial and domestic environments and also in public utilities. For surfaces having and not having contact with food. Designed also for disinfection of non-invasive medical devices. The product has a bactericidal activity, yeasticidal and limited virucidal activity (against Rotavirus, Norovirus), it also works against tuberculosis and enveloped viruses (including Vaccinia, BVDV, HIV, HBV, HCV, influenza and coronavirus). As a medical device for use by a professional user.

Effectiveness	EN 14885	Conditions	
		clean	dirty
Bacteria (including MRSA)	EN 13727	30 sec.	30 sec.
Fungi (C. albicans)	EN 13624	15 sec.	15 sec.
Tuberculosis (M. terrae)	EN 14348	30 sec.	30 sec.
Enveloped viruses (Vaccinia, HIV, HBV, HCV, SARS-Cov-2)	EN 14476	30 sec.	60 sec.
BVDV virus	EN 14476	30 sec.	30 sec.
Rota virus	EN 14476	30 sec.	60 sec.
Noro virus	EN 14476	30 sec.	30 sec.
Hygienic hand disinfection	EN 1500	2 x 3 ml in 2 x 15 sec.	
Surgical hand disinfection	EN 12791	2 x 3 ml in 2 x 90 sec.	

Directions for use:

Hygienic hand disinfection (according to EN 1500): 2 x 3 ml of the product to be rubbed for 30 seconds (2 x 15 sec.).

Surgical hand disinfection (according to EN 12791): 2 x 3 ml of the product to be rubbed for 3 minutes (2 x 90 sec).

Keep hands equally wet by adding more liquid.

Surface disinfection: Spray the surface from the distance of approximately 30 cm and leave it to dry. Do not apply on alcohol-sensitive surfaces (e.g. acrylic glass). For professional use only.

Ingredients: active substance in 100 g of the product:

63.7 g ethanol, 6.3 g propan-2-ol

Packaging units:

250ml bottle,
500ml bottle,
1l bottle,
1l bag



Permission number for biocidal products in Poland: 7674/19

The safety data sheet and the component data sheet are available at www.medisep.pl

CE 2274